

recliner operating instructions



knú
CONTRACT


L A Z B O Y
CONTRACT FURNITURE

RECLINER FOOTREST

The handle to operate the footrest cannot be used while the tray is in the folded down position.

The patient will not be able to operate the handle when the tray is in the upright position.

The tray will block their ability to reach the handle to open the footrest.

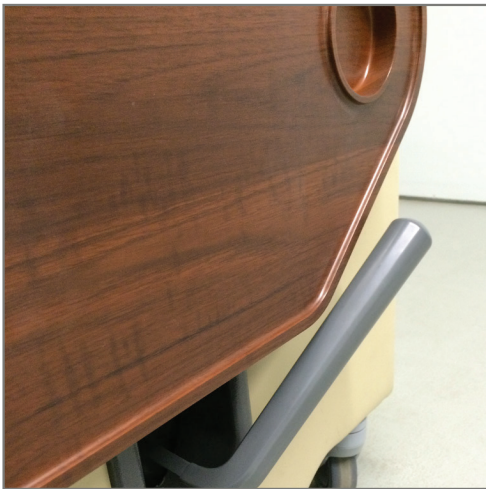

In order to open the footrest, the caregiver will have to operate the handle.



RECLINER FOOTREST

Pull the cable handle out while pushing back with your recline.

Release the handle when the desired angle of recline is achieved.

The back will lock in an infinite number of positions.



Pull the cable handle out and lean weight forward in chair to return the back to the upright position.



RECLINER FOOTREST

Push down with legs to close the footrest/kickout.

